GERSON®

3230 - N95 RESPIRATOR Display Box FRONT

GERSON®

16 Commerce Blvd.
Middleboro, MA 02346
Tel 508.947.4000 Fax 508.947.5442
sales@gersonco.com • www.GersonCo.com
© 2020 Louis M. Gerson Co., Inc. All Rights Reserved

GERSON®

3230 - N95 RESPIRATOR Display Box BACK

Respirator Fitting Instructions:

1. Unwrap respirator and bend nosepiece into a curve to open mask.
2. Separate the pre-scored elastic headband into two individual bands.
3. Hold the headbands with your index fingers and thumbs.
4. Cup the open mask securely under your chin.
5. Pull both headbands over your head.
6. Rest the top headband on the crown of your head and bring the bottom one down under your ears.
7. Press the nosepiece around the contour of your nose and cheekbones. Adjust the edges of the mask to make sure there are no gaps.
8. Perform a Fit Check by gently cupping both hands around the mask, then exhale sharply to see if you sense or detect any leaks.

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WARNINGS

Read and understand warnings on insert before using.

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100% polypropylene
100% polypropylène
100% polipropileno

NATIONAL INSTITUTE FOR OCCUPATIONAL SAFETY & HEALTH

GERSON®

WARNING: A respirator does not eliminate the risk of contracting any disease or infection. Occupational use of this respirator requires that you be fit tested by a qualitative or quantitative fit test and that your employer comply with OSHA 29 CFR 1910.134, which includes medical evaluation, training and fit testing.

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0 44585 32301 7

GERSON®

LOUIS M. GERSON CO., INC.
Middleboro, MA 02346 U.S.A.
www.gersonco.com
© 2020, All Rights Reserved
Made in U.S.A.

GERSON®

N95 MODEL 3230 RESPIRATOR

Typical 3230 Applications:

VIRUSES
CUTTING
DEMOLITION
SANDING, GRINDING
MOLD REMEDIATION
DUST & POLLENS

Catalog # 083230

Made in U.S.A.

GERSON®

16 Commerce Blvd.

Middleboro, MA 02346

Tel 508.947.4000 Fax 508.947.5442

sales@gersonco.com • www.gersonco.com

© 2020 Louis M. Gerson Co., Inc. All Rights Reserved