Perforate

Fold

Cut

Glue/Adhesive

**Respirator**  
1730  
Disposable Surgical N95 Respirator

Cat. No. 081730

**1730**  
**Respirator**  
Disposable Surgical N95 Respirator

**FLUID RESISTANT**  
**MEETS CDC GUIDELINES**  
**FOR TB EXPOSURE CONTROL**  
For Single Use Only  
Solamente para usarse una vez.

**GERSON** 20

55488ea-Person1730\_DisigBox

**Respirator**  
1730  
Disposable Surgical N95 Respirator

**WARNINGS**  
Read and understand before use.

- A properly selected respirator is essential to protect your health. Before using this or any product consult an Industrial Hygienist or Occupational Safety Professional to determine its suitability for your intended use.
- This product will not protect against gases or vapors. Do not use in spray paint operations.
- Do not use in sand blasting operations.
- Do not use for protection against asbestos.
- This product does not supply oxygen. Use this respirator in a well ventilated area containing sufficient oxygen to support life.
- Do not use when concentrations of contaminants are unknown or immediately dangerous to life.
- Leave work area immediately if: (A) breathing becomes difficult, or (B) dizziness or other signs of distress occur.
- If user has any illness or disorders, consult a licensed medical doctor to determine suitability of use.
- Facial hair and certain facial characteristics may prohibit effective use of this product.
- Use only in accordance with instructions, labels and limitations relating to this product.
- Do not alter or modify this product in any way.
- Follow the manufacturer's instructions for fitting the respirator to the face.
- Filtering facepieces are to be inspected prior to each use to assure there are no holes in the breathing zone and that no damage has occurred.

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- Hold the respirator in hand with nosepiece at your fingertips, allowing the headbands to hang freely below your hand.
- Press the respirator firmly against your face with the nose-piece on the bridge of your nose.
- Stretch and position the top band high on the back of your head. Stretch the bottom band over the head and position it below your ears.
- Using both hands, mold the metal nosepiece to the shape of your nose.
- To test fit cup both hands over the respirator and exhale vigorously. If air flows around your nose, lighten the nosepiece, if air leaks around the edges, reposition the straps for a better fit.
- Remember, careful observance of these fitting instructions is an important step in safe respirator use.

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**IMPORTANT NOTICE TO PURCHASER**  
Manufacturer warrants this product to be free from defects in material or workmanship. In the event that this product proves upon inspection to contain defects in material or workmanship, Manufacturer undertakes, at its option, to replace any portion proving defective or to refund to the Buyer the purchase price paid by Buyer for this product. This warranty and Buyer's remedy of replacement or refund constitute the Manufacturer's exclusive warranty and the Buyer's sole remedy, in lieu of all other warranties expressed or implied. Except as provided above, Manufacturer shall not be liable for any loss or damage, whether direct, consequential, or incidental occasioned by the sale, use or misuse of this product, other than death or personal injury directly resulting from Manufacturer's negligence. Buyer shall determine the suitability of this product for its intended use and except provided above, assumes all risks and liability in connection therewith.

**S-Special or Critical User Instructions**  
This respirator has been approved as a NIOSH N95 filtering facepiece respirator for use in healthcare settings, as a Surgical N95 Respirator conforming to recognized standards for biocompatibility, flammability, and fluid resistance.

**LOUIS M. GERSON CO., INC.**  
Middleboro, MA 02346 U.S.A.  
© 2022, All Rights Reserved  
Made in U.S.A.  
[www.GersonCo.com](http://www.GersonCo.com)

**Louis M. Gerson Co., Inc.**  
MIDDLEBORO, MA 02346  
1-800-225-9823

**NIOSH**  
NATIONAL INSTITUTE FOR OCCUPATIONAL SAFETY AND HEALTH

This respirator is approved only in the following configuration.

TC-	Protection <sup>1</sup>	Respirator	Cautions and Limitations <sup>2</sup>
		1730	
84A-0160	Surgical/N95	X	ABC,IMND5

**1. Protection**  
N-95 Particulate filter ( 95% filter efficiency level ) effective against particulate aerosols free of oil; time use restrictions apply.

**2. Cautions and Limitations**

- A - Not for use in atmospheres containing less than 19.5% oxygen.
- B - Not for use in atmospheres immediately dangerous to life or health.
- C - Do not exceed maximum use concentrations established by regulatory standards.
- J - Failure to properly use and maintain this product could result in injury or death.
- M - All approved respirators shall be selected, fitted, used and maintained in accordance with MSHA, OSHA, and other applicable regulations.
- N - Never substitute, modify, add or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.
- O - Refer to user instructions and/or maintenance manuals for information on use and maintenance of the respirators.
- S - Special or Critical User's Instructions and/or specific use limitations apply. Refer to User's Instructions before donning.

**See Information Supplied** **Temperature Range of Storage Conditions** **Maximum Humidity of Storage Conditions**

0.625in  
15.875mm

5.25in  
133.35mm

5.25in  
133.35mm

5.22in  
132.588mm

1.25in  
31.75mm

1.75in  
44.45mm

6.6250in  
168.275mm