Perforate

Fold

Cut

Glue/Adhesive

0.168in
4.2672mm

6.6856in
169.81424mm

0.625in
15.875mm

0.625in
15.875mm

5.25in
133.35mm

5.25in
133.35mm

5.25in
133.35mm

5.22in
132.588mm

1.25in
31.75mm

1.75in
44.45mm

7.50in
190.50mm

Crosstex International, Inc.
6789 W. Henrietta Road, Rush, NY 14543 USA
www.crosstex.com
crosstex@crosstex.com

This respirator is approved only in the following configuration.

TC	Protection ¹	Facepiece	Cautions and Limitations ²
84A-0160	Surgical/N95	X	ABC, JM, NOS

- Protection
 - N95 Particulate filter (95% filter efficiency level) effective against particulate aerosols free of oil; time use restrictions may apply.
- Cautions and Limitations
 - A - Not for use in atmospheres containing less than 19.5% oxygen.
 - B - Not for use in atmospheres immediately dangerous to life or health.
 - C - Do not exceed maximum use concentrations established by regulatory standards.
 - J - Failure to properly use and maintain this product could result in injury or death.
 - M - All approved respirators shall be selected, fitted, used and maintained in accordance with NIOSH, OSHA and other applicable regulations.
 - N - Never substitute, modify, add or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.
 - O - Refer to user instructions and/or maintenance manuals for information on use and maintenance of the respirators.
 - S - **Special or Critical User's Instructions and/or specific use limitations apply. Refer to User's Instructions before donning.**

IMPORTANT NOTICE TO PURCHASER
Manufacturer warrants this product to be free from defects in material and workmanship. In the event that this product proves upon inspection to contain defects in material or workmanship, Manufacturer undertakes, at its option, to replace any portion proving defective or to refund to the Buyer the purchase price paid by Buyer for this product. This warranty and Buyer's remedy of replacement or refund constitute the Manufacturer's exclusive warranty and the Buyer's sole remedy, in lieu of all other warranties expressed or implied. Except as provided above, Manufacturer shall not be liable for any loss or damage, whether direct, consequential, or incidental occasioned by the sale, use or misuse of this product, other than death or personal injury directly resulting from Manufacturer's negligence. Buyer shall determine the suitability of this product for its intended use and except provided above, assumes all risks and liability in connection therewith.

Distributed by / Distributeur par / Distribuidor por
CROSSTEX
Crosstex International, Inc.
6789 W. Henrietta Road, Rush, NY 14543 USA
585.389.0100 | 631.582.6771 | crosstex.com

Single Use Only
Not tested for reprocessing

GMLRESP BX / REV C - 010022
All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC, its affiliates or related companies, unless otherwise noted.
©2022 Hu-Friedy Mfg. Co., LLC. All rights reserved.

REF: GMLRESP

(01)10732224004690

PI Barcode with LOT information

EN

Warnings

- Read and understand before use.
- A properly selected respirator is essential to protect your health. Before using this or any product consult an Industrial Hygienist or Occupational Safety Professional to determine its suitability for your intended use.
- This product will not protect against gases or vapors.
- Do not use in spray paint operations.
- Do not use in or around sand blasting operations.
- Do not use for protection against asbestos.
- This product does not supply oxygen. Use this respirator in a well-ventilated area containing sufficient oxygen to support life.
- Do not use when concentrations of contaminants are unknown or immediately dangerous to life.
- Leave work area immediately if:
 - a) Breathing becomes difficult, or
 - b) Dizziness or other signs of distress occur.
- User has any illness or disorders, consult a licensed medical doctor to determine suitability of use.
- Facial hair and certain facial characteristics may prohibit effective use of this product.
- Use only in accordance with instructions, labels and limitations relating to this product.
- Do not alter or modify this product in any way.
- Follow the manufacturer's instructions for fitting the respirator to the face.

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**Crosstex™ Isolator™
Molded Surgical
N95 Respirator**

**Surgical N95 Respirator
Respirateur chirurgical N95
Respirador quirúrgico N95**

20 per box REF: GMLRESP

EN

Respirator fitting instructions:

- Hold the respirator in hand with nosepiece at your fingertips, allowing the headbands to hang freely below your hand.
- Press the respirator firmly against your face with the nosepiece on the bridge of your nose.
- Stretch and position the top band high on the back of your head. Stretch the bottom band over the head and position it below your ears.
- Using both hands, mold the metal nosepiece to the shape of your nose.
- Perform a seal check by gently cupping both hands around the respirator. Then inhale sharply to see if you sense or detect any leaks. If leaks are detected, reposition the respirator and/or adjust the straps. If leaks are still detected, then do not enter contaminated areas and seek supervisor for further instructions.
- Remember, careful observation of these fitting instructions is an important step in safe respirator use.

NOTE: Before occupational use of this respirator a written respiratory protection program must be implemented meeting all the local government requirements. In the United States employers must comply with OSHA 29 CFR 1910.134 which includes medical evaluation, training, and fit testing.
NOTE: Before issuing any respirator to be worn, fit test should be performed.

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S - SPECIAL OR CRITICAL USER INSTRUCTIONS:

- This respirator has been approved as a NIOSH N95 filtering facepiece for use in healthcare settings as a Surgical N95 Respirator conforming to recognized standards for biocompatibility, flammability, and fluid resistance at 160mmHg.
- This model of filtering facepiece has been manufactured for Crosstex International, Inc. by the Louis M. Gerson Co., Inc. under approval TC-84A-0160.

White Spot Color
Pantone Process Black
CMYK 0/0/0/100