Crosstex™ Isolator™ Duckbill Surgical N95 Respirator

Manufactured by:
Crosstex International, Inc.
6789 W. Henrietta Road, Rush, NY 14543 USA
www.crosstex.com
crosstex@crosstex.com

These respirators are approved only in the following configurations:

TC-	Protection ¹	Respirator	Cautions and Limitations ²
		GDBRESP	
84A-9317	Surgical/N95	X	ABCJMNS

¹Protection

N95 Particulate filter (95% filter efficiency level) effective against particulate aerosols free of oil; time use restrictions may apply.

²Cautions and Limitations

- A - Not for use in atmospheres containing less than 19.5% oxygen.
- B - Not for use in atmospheres immediately dangerous to life or health.
- C - Do not exceed maximum use concentrations established by regulatory standards.
- J - Failure to properly use and maintain this product could result in injury or death.
- M - All approved respirators shall be selected, fitted, used and maintained in accordance with MSHA, OSHA and other applicable regulations.
- N - Never substitute, modify, add or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.
- O - Refer to user instructions, and/or maintenance manuals for information on use and maintenance of the respirators.
- S - Special or Critical user's instruction user's instructions and/or specific use limitation apply. Refer to user instruction before donning.

EN IMPORTANT NOTICE TO PURCHASER

Manufacturer warrants this product to be free from defects in material and workmanship. In the event that this product proves upon inspection to contain defects in material or workmanship, Manufacturer undertakes, at its option, to replace any portion providing defective or to refund to the Buyer the purchase price paid by the Buyer for this product. This warranty and Buyer's remedy of replacement or refund constitute the Manufacturer's exclusive warranty and the Buyer's sole remedy in lieu of all other warranties expressed or implied. Except as provided above, Manufacturer shall not be liable for any loss or damage, whether direct, consequential, or incidental occasioned by the sale, use or misuse of this product, other than death or personal injury directly resulting from Manufacturer's negligence. Buyer shall determine the suitability of this product for its intended use and except provided above, assumes all risks and liability in connection therewith.

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HuFriedyGroup
The Best In Practice

S-Special or Critical User Instructions

- This respirator has been approved as a NIOSH N95 filtering facepiece respirator for use in healthcare settings, as a Surgical N95 Respirator conforming to recognized standards for biocompatibility, flammability, and fluid resistance.
- The model GDBRESP has been manufactured by Louis M. Gerson CO., Inc. for private label Crosstex International, Inc. under TC-84A-9317.

EN

WARNINGS

Read and understand before use.

- A properly selected respirator is essential to protect your health. Before using this or any product consult an Industrial Hygienist or Occupational Safety Professional to determine its suitability for your intended use.
- A respirator does not eliminate the risk of contracting any disease or infection.
- This product will not protect against gases or vapors.
- Do not use in spray paint operations.
- Do not use in or around sand blasting operations.
- Do not use for protection against asbestos.
- This product does not supply oxygen. Use this respirator in a well ventilated area containing sufficient oxygen to support life.
- Do not use when concentrations of contaminants are unknown or immediately dangerous to life.
- Leave work area immediately if:
 - a) breathing becomes difficult, or
 - b) dizziness or other signs of distress occur.
- If user has any illness or disorders, consult a licensed medical doctor to determine suitability of use.
- Facial hair and certain facial characteristics may prohibit effective use of this product.
- Use only in accordance with instructions, labels and limitations relating to this product.
- Do not alter or modify this product in any way.
- Follow the manufacturer's instructions for fitting the respirator to the face.

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NOTE: Before occupational use of this respirator, a written respiratory protection program must be implemented meeting all the local government requirements. In the United States, employers must comply with OSHA 29 CFR 1910.134 which includes medical evaluation, training and fit testing.

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GDBRESP-INS / REV A - 09/2021

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC, its affiliates or related companies, unless otherwise noted.

©2021 Hu-Friedy Mfg. Co., LLC. All rights reserved.

Single Use Only
Not tested for reprocessing
Not made with natural rubber latex

See information supplied by the manufacturer.

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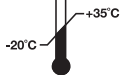
End of shelf life

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Da usari entro

Temperature range of storage conditions.

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Maximum humidity of storage conditions.

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